



How to Use the QuickScan 650 Scanner

1. Remove the plastic wrapping and insert batteries prior to use.
2. Continuously press and hold the red button to turn the scanner on – it will beep and the LCD screen will light up when the sensors are activated. If the scanner has been used before, it will display the number of the last microchip read.
 - Still pressing the red button, hold the scanner with the red “scanning” sensor pointed down toward the pet. Slowly scan the area between the shoulder blades in a zigzag motion.
 - If no microchip is detected between the shoulder blades, scan a broader area, including the front legs down to the elbows and the neck.
3. When a microchip is detected, the scanner will beep and the screen will display the pet’s microchip number.
4. To help reunite a found pet with its owner, go to www.petmicrochiplookup.org or call AKC Reunite at 800-252-7894.

The scanner will automatically turn itself off approximately 30 seconds after you stop pressing the red button.

For future reference: if the scanner is working slowly, replace the 9 volt battery.



This scanner comes with a twelve-month manufacturer’s warranty. Please do NOT write on, mark on or otherwise damage or deface this scanner as this may void the warranty.